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User manual Dynair® ANIMAL Pad

Congratulations!

You are now the owner of a Dynair® ANIMAL Pad a product “made in Germany”. This sports tool was produced in compliance with valid EU norms, it contains no heavy metals and is made of 100% recyclable plastic.

The TOGU Dynair Animal Pad is a versatile, air-filled training cushion for canine physiotherapy and dog sports. It can be used on both sides and allows for variable training intensity thanks to its adjustable air pressure.

Warning:

- Avoid storage in direct sunlight or near heat sources, because otherwise Dynair® ANIMAL Pad might deform
- Avoid contact with sharp and pointed objects (e.g. broken glass) because they can damage the Dynair® ANIMAL Pad. Do not throw the Dynair® ANIMAL Pad!
- Prior to any use, the Dynair® ANIMAL Pad should be checked for damage because otherwise a safe handling is not guaranteed
- The Dynair® ANIMAL Pad has to rest on a horizontal, stable and dry ground, to make sure, that the Dynair® ANIMAL Pad cannot slide away during the exercises.

Equipment details:

The Dynair® ANIMAL Pad is made of high-grade Ruten with a needle valve.

Size: 32,5 cm x 32,5 cm x 8 cm

Load: 150 kg

Care:

Use mild standard detergents and disinfectants. Dissolvers as well as acidic or corrosive cleansers can harm the material. The Dynair® ANIMAL Pad is waterproof

How to inflate:

The air filling of the Dynair® ANIMAL Pad can be regulated by using a standard needle pump (e.g. ball pump)

Therefore, the Dynair® ANIMAL Pad is equipped with a needle valve.

Please perform changes to the air pressure inside the Dynair® ANIMAL Pad (inflation or bleeding) only, when the Dynair® ANIMAL Pad has room temperature and add at the most as much air, as is necessary to make the surface level. Always moisten the needle before carefully inserting into the needle valve.