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## Instructions for use and care: HORSE BALANCE Pad

Please read the following safety instructions and only use the product as described to avoid injury and damage. Keep these instructions for future reference. Be sure to include the instructions when passing the product on to someone else.

Congratulations on purchasing your new HORSE BALANCE Pad.

With the HORSE BALANCE Pad, you have chosen a TOGU® brand product **“made in Germany”**!

The special balance pad with innovative gel filling. Can be used as a reactive-dynamic base according to the TOGU ANIMAL concept, for joint stabilizing as well as for coordination and sensorimotor exercises for horses.

### Features:

Both sides with a velvety smooth surface: HORSE BALANCE Pad

The HORSE BALANCE Pad is filled with a water-based gel. The outer cover is made of high-quality Ruton®.

Diameter: approx. 30 cm

Weight: ca. 3,9kg (8,6 lbs)

Load capacity: max. 1.000 kg (2.200 lbs)

### Temperature resistance:

The HORSE BALANCE Pad is UV-resistant and also heat-resistant to a certain extent. The HORSE BALANCE Pad can be used without any problems at temperatures up to 45°C. It should not be stored in direct sunlight. The material becomes softer as the temperature rises.

The HORSE BALANCE Pad is also cold-resistant: the storage temperature should not fall below -10°C. The material becomes harder as the temperature drops.

### Care:

You can use any mild, commercially available cleaning agent or disinfectant to clean the HORSE BALANCE Pad. Solvent-based, acidic, or corrosive cleaners will damage the material. The HORSE BALANCE Pad is waterproof.

### Storage:

Please always store your HORSE BALANCE Pad flat.

### ATTENTION!

#### Important safety information:

- The product is not suitable for small children! It is not a toy!
- The sealing plug must not be removed.
- Keep the device away from sharp or pointed objects to avoid damage to the device.
- Leaky/damaged devices must no longer be used. In this case, you can send the device to us for recycling.
- The product contents are not intended for consumption.