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## Instruction for use: Relax Nex® Thermo

**Congratulations on your purchase of the Relax Nex® Thermo. You have chosen a TOGU branded product "Made in Germany".**

The **Relax Nex® Thermo** is a new hot-cold pillow with special thermo-gel filling. Due to the pleasant - 100% skin-friendly - design it adapts to your body during the application. The use is recommended especially for the neck, shoulders and back. In the heated state it can relax muscles tension and create well-being. Ideal after a long day's work or relaxation in between. In a cold state, the pillow can support the regeneration after exercising and sports. The Relax Nex® Thermo is made of Ruton® extremely resistant (load capacity up to 200kg) and durable. It is easy to clean, hygienic and odorless. Enjoy relaxing with your new Relax Nex® Thermo.

Read the following safety instructions and use the product only as described to avoid injuries or damages. Keep this manual for later reference. Forward the instructions to every user.

### Please note when heating up:

Please note that the surface of the cushion will heat up even further after removing it from the water bath or microwave. **This process may take several minutes.** To avoid skin burns, make sure before each use that the gel cushion is not damaged and test the temperature gently on the forearm. If the gel cushion is too hot, please let it cool down slightly.

1: in warm water bath at **max. 60°C (140°F)**

2: in a microwave one time for max. **4 minutes at max. 700 watts**. Do not repeat the heating procedure before 2 hours have passed. Do not use the microwave, if it is not adjustable to 700 watts.

### Please note when cooling down:

Use a refrigerator to cool down the Relax-Nex® Thermo with a temperature not less than 3°C (37°F). Do not put it into the freezer!

### ATTENTION! Important safety instructions:

- Not suitable for small children! It is not a toy!
- Do not remove the plug.
- The gel cushion must not be used by persons with disturbed perception of heat and cold (children under 3 years, disabled people). Children over 3 years can only use it under supervision.
- Quit the application, if the heat / cold is perceived as unpleasant.
- The use of heat should be generally avoided in case of febrile diseases, eczema, heart and vein diseases, tumors or cancer.
- To avoid burns and possible skin irritation, put the heated gel cushion only gently on the skin, or alternatively use a cloth between cushion and skin.
- The gel cushion must not be heated in an oven. Explosion risk!
- Be sure to follow the specified heating instructions above! Longer heating times or a higher wattage may lead to overheating of the gel cushion and cause damage.
- Watch the gel cushion during heating procedure in the microwave. When the gel cushion starts to inflate stop the procedure immediately! Then, after cooling, check it for tightness and choose a shorter heating time.
- Be careful when removing the gel cushion out of the microwave! Overheating can destroy it and cause skin burns. - Pointed or sharp objects can damage the product.
- Leaking or damaged gel cushion may no longer be used.
- The gel content is not intended to be eaten.

### Care instruction:

To clean the Relax-Nex® Thermo you can use any mild standard cleaning and disinfectants. Solvent-containing, acidic or caustic cleaners will attack the material. The Relax-Nex® Thermo is waterproof. Storage at room temperature recommended.