**MD Aero-Step® XL**  
**MD Aero-Step® Pro**

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**TOGU GmbH**

Atzinger Str. 1

D-83209 Prien-Bachham

Tel.: +49 (0)8051 9038-0

info@togu.de

Version 3/22

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**Medical device class 1**

## Table of contents

<b>1</b>	<b>General information</b> .....	<b>3</b>
1.1	Definitions .....	3
1.2	Abbreviations .....	5
<b>2</b>	<b>Safety</b> .....	<b>6</b>
2.1	Safety instructions .....	6
2.2	Product lifetime .....	6
2.3	Remaining residual risks .....	6
2.4	Cautions/Warnings .....	6
<b>3</b>	<b>Device description</b> .....	<b>7</b>
3.1	Intended use .....	7
3.2	Device description .....	8
3.3	Performance description .....	8
3.3.1	Product performance .....	8
3.3.2	Product lifetime .....	9
<b>4</b>	<b>Application</b> .....	<b>9</b>
4.1	Commissioning .....	9
4.2	Correct use .....	9
4.3	When should the product not be used?.....	10
<b>5</b>	<b>Cleaning and disinfection</b> .....	<b>10</b>
<b>6</b>	<b>Storage and transport</b> .....	<b>10</b>
<b>7</b>	<b>Waste disposal</b> .....	<b>11</b>
<b>8</b>	<b>Legal notice</b> .....	<b>11</b>

# 1 General information

Do not use the device until you have read and understood the operating instructions.

All information contained in these instructions for use is the property of **TOGU GmbH** (= manufacturer) and may not be reproduced or duplicated without prior permission. The manufacturer reserves the right to change or completely replace this instruction manual at any time and without prior notice.

You should always ensure that you are working with the latest version of the operating instructions. If there is any doubt about the up-to-dateness, contact the manufacturer.

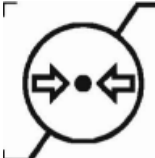
Keep these instructions for use in a safe place and pass them on to other users of the product before use.

TOGU® and MD Aero-Step® are protected word marks of Gebr. Obermaier oHG.

## 1.1 Definitions

In these operating instructions, special information is marked as follows:

Symbol	Meaning
	<p>Manufacturer</p> <p>In addition to this symbol, the name and address of the manufacturer, i.e. the person who places the (medical) device on the market, must be indicated</p>
	<p>Date of manufacture</p> <p>Next to this symbol, the date of manufacture of the (medical) product must be indicated in the form YYYY-MM, e.g. 2019-12 for December 2019.</p>
	<p>Usable until</p> <p>For (medical) products that are restricted in their period of use, this symbol shows the date after which the (medical) product may no longer be used in the form YYYY-MM-DD, e.g. 2020-12-01 for 01.12.2020. The date must appear next to the symbol.</p>
	<p>Production lot number/batch</p> <p>Indicates the lot number to which the respective (medical) product belongs. It consists of a 6-digit code and indicates the date of manufacture in the format YYMMDD (year, month, day, each with 2 digits). The lot number is located on the packaging and must be next to the symbol.</p>
	<p>Article number/product identification number</p> <p>Specifies the article or product identification number that must be next to the symbol.</p>

Symbol	Meaning
	<p>Protect from sunlight!</p> <p>If the (medical) product and/or its packaging can be negatively affected in its properties by sunlight, this symbol must be applied to the packaging as a warning.</p>
	<p>Store in a dry place.</p> <p>If the (medical) product and/or its packaging may be negatively affected in its properties by direct moisture, this symbol must be applied to the packaging as a warning.</p>
	<p>Temperature limit.</p> <p>If the (medical) product and/or its packaging can be negatively affected in its properties by certain temperature ranges, this symbol with indication of the lower and upper temperature limits must be affixed to the packaging as a warning.</p> <p>The limit values must be indicated in each case next to the horizontal lower or upper line.</p>
	<p>Air humidity limit.</p> <p>If the (medical) product and/or its packaging can be negatively influenced in its properties by certain air humidity ranges, this symbol with indication of the lower and upper air humidity limits must be affixed to the packaging as a warning.</p> <p>The limit values must be indicated in each case next to the horizontal lower or upper line.</p>
	<p>Air pressure limit.</p> <p>If the (medical) product and/or its packaging can be negatively influenced in its properties by certain air pressure ranges, this symbol with indication of the lower and upper air pressure limits must be affixed to the packaging as a warning.</p> <p>The limit values must be indicated in each case next to the horizontal lower or upper line.</p>
	<p>Observe the instructions for use!</p> <p>If prior reading of the operating instructions is required for correct use of the (medical) product, this symbol must be clearly visible.</p>
	<p>"Reference to eIFU" shall be replaced with the website address where the user can find the electronic user documentation.</p> <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;">   <b>Hinweis auf eIFU</b> </div> <div style="text-align: center;">   <b>Hinweis auf eIFU</b> </div> <div style="text-align: center;">   <b>Hinweis auf eIFU</b> </div> </div>
	<p>Attention!</p> <p>If it is absolutely necessary to read the operating instructions before using the (medical) product without risk, this symbol must be clearly visible.</p>
<p>Max. xxx kg</p>	<p>Performance information relevant for the safe use of the product</p>
	<p>MD = Medical Device</p> <p>Marking that the product is a medical device.</p>

Symbol	Meaning
	<p>CE conformity marking (minimum height 5mm).            Indicates compliance with the Essential Requirements of the applicable European Directive/Regulation.            For Class I medical devices, only the two letters are affixed.            For medical devices &gt; Class I, the two letters are followed by the four-digit number of the Notified Body that carried out the conformity assessment procedure.</p>

## 1.2 Abbreviations

Shortcut	Meaning
JJMMTT	Year (2 digits), Month (2 digits), Day (2 digits)

## 2 Safety

### 2.1 Safety instructions

### 2.2 Product Lifetime

If the service life of 2 years from first use, or at the latest 10 years after manufacture (see lot number) is exceeded, safe use of the product is no longer guaranteed. Replace the product immediately.

### 2.3 Remaining residual risks

Please keep the exercise field free of sharp objects or edges. Before each use of your product, check that the surface is undamaged. Product safety is no longer guaranteed if the surface is damaged or if the safety limits described in this manual are exceeded.

### 2.4 Cautions/Warnings

The product must not be misused, modified or changed in any other way. Safe use is no longer guaranteed in this case.

Only use the product on a stable, horizontal, dry and clean surface. In this way, prevent the product from slipping away.

Keep the product away from:

- sharp and pointed objects
- fire or other sources of heat
- Chemicals

Do not wear clothing or shoes with sharp or pointed parts while using the product.

Keep the area around the product free of objects (furniture, fixtures) within a radius of the user's body length. Keep your distance from other persons. This will prevent injuries in the event of a fall.

The product is unstable. You may suddenly lose your balance. Move onto the product carefully and slowly. Use safe, firmly anchored aids to avoid a fall and injuries.

Sweating or moisture from outside (e.g. rain) can make the product slippery. Also check the surface of the product during use. In this way, avoid slipping and injury in the event of a fall.

The product is water resistant. However, you should ensure that the product is dry before use, as a wet undersurface may cause the MD Aero-Step® XL / Pro to slip.

The product should be used at room temperature. The product can be used in a temperature range from **-10°C to +45°C**. It must not be used in direct sunlight (e.g. by the window) or near any other heat source (e.g. by the heater).

The material is thermoplastic: this means that it becomes softer with increasing heat and firmer with increasing cold.

Do not wear clothing or shoes with sharp or pointed parts while using the product.

Keep the area around the product free of objects (furniture, fixtures) within a radius of the user's body length. Keep your distance from other persons. This will prevent injuries in the event of a fall.

Check the product for damage, wear, and functionality before use. If there are any deviations, the product must no longer be used. Remove and replace the product if it shows signs of damage. Do not attempt to repair the product.

Do not wear clothing or shoes with sharp or pointed parts while using the product. Keep the area around the product free of objects (furniture, fixtures) within a radius of the user's body length. Keep your distance from other persons. This will prevent injuries in the event of a fall.

Check the product for damage, wear, and functionality before use. If there are any deviations, the product must no longer be used. Remove and replace the product if it shows signs of damage. Do not attempt to repair the product.

In these cases the product should not be used: see section 4.3

### 3 Device description

MD Aero Step® XL / MD Aero-Step® Pro

With its 2-chamber air system, the MD Aero-Step® XL / Pro enables training of joint stability and can tone and strengthen muscles deep down. Equipped with over 1,000 round knobs, the surface has a massaging effect and promotes blood circulation.

It can be used while standing or in the prone or supine position. The lability created by the air filling produces the desired training effect.

The MD Aero-Step® XL / Pro is made of high quality ruton, which gives the product a long lifetime.

The product is water resistant. However, you should ensure that the product is dry before use, because a wet undersurface can cause the MD Aero-Step® XL / Pro to slip.

The material used can be recycled several times, which is why a defective product represents a high-quality raw material. Please recycle a defective product in order to make full use of ecological resources.

Our products comply with the German Foodstuffs and Commodities Act (BGVV) and do not contain any heavy metals.

In addition, we guarantee that our products are free of harmful or odorous vapors.

#### 3.1 Intended use

The MD Aero-Step® XL / Pro is a medical device and is used as a non-active therapeutic device for:

- Diseases of the musculoskeletal system,
- as prophylaxis in the professional field and
- as prophylaxis in the private/amateur sector.

The product is used by physiotherapists (or similar qualification) in rehabilitation or physiotherapy facilities (or similar), as well as by patients (age from 6 years) in the home environment.

In therapy, the MD Aero-Step® XL / Pro is used, among other things, in orthopedics and neurological rehabilitation to train mobility, strength and coordination and also serves to promote sensory function, coordination and the musculoskeletal system.

#### Contraindications:

The MD Aero-Step® XL / Pro has been developed for various therapeutic and sporting applications and must not be used other than for the specified purpose.

The use of the product is prohibited in case of:

- Severe osteoporosis
- Other diseases that greatly reduce the strength of the bony system

Conditional contraindications are defined as follows (only after consultation with physician/therapist):

- Balance disorders
- Dizziness
- Epilepsy

Do not train with weights or dumbbells of any kind on the MD Aero-Step® XL / Pro.

### **3.2 Device description**

<b>Description</b>	<b>Size</b>	<b>Weight net</b>
MD Aero-Step® XL	51x37x8 cm	2.800 g
MD Aero-Step® Pro	52x40x8,5cm	3.600 g

Included in the delivery is a package, quick start guide, instructions with examples of use.

### **3.3 Performance description**

In the table below, the load capacity is specified in relation to the maximum load capacity, i.e. how much weight an MD Aero-Step® XL / Pro can safely carry as a maximum in undamaged condition (max. load).

#### **3.3.1 Product performance**

<b>Description</b>	<b>max.load</b>
MD Aero-Step® XL	ca. 200 kg
MD Aero-Step® Pro	ca. 200 kg

### 3.3.2 Product lifetime

If the service life of the product exceeds 2 years from the date of first use, or a maximum of 10 years from the date of manufacture (see lot number), safe use of the product is no longer guaranteed. Replace the product immediately.

## 4 Application

### 4.1 Commissioning

When opening the packaging, take care not to damage the product.

Check the product for damage, wear and functionality before use. If there are any deviations, the product must no longer be used. Remove and replace the product if it shows any damage. Do not attempt to repair the product.

### 4.2 Correct use

Observe the detailed safety instructions under point 2.

The product should be used at room temperature. The product can be used in a temperature range from **-10°C to +45°C**. Keep the area around the product free of objects (furniture, fixtures, etc.) within a radius of the user's body length. Keep a distance from other persons. This will prevent injury in the event of a fall.

Before each use, check the product for damage, wear and tear and proper functioning. If there are any deviations, the product must no longer be used. Remove and replace the product if it shows signs of damage.

You should avoid using the product on a stony surface (gravel, pebbles), as well as asphalt. Avoid pointed objects or sharp edges in the area of use, as these could injure the MD Aero-Step® XL / Pro.

If the service life of the product exceeds 2 years from the date of first use, or a maximum of 10 years from the date of manufacture (see lot number), safe use of the product is no longer guaranteed. Replace the product immediately.

For the application, please note:

- Adopt a correct posture
- Perform slow and controlled movements
- If pain occurs, seek advice from a doctor or therapist one
- Children are allowed to train only under the supervision of adults

### **Inflate:**

The MD Aero-Step® XL / Pro is delivered optimally filled with air.

You should always make sure that the surface is "flat".

If necessary, you can subsequently regulate the air volume with a needle valve pump (e.g. the TOGU® pump or with any conventional soccer pump).

Please make sure that the needle does not poke into the lower side of the MD Aero-Step®. Always wet the needle of the pump before carefully inserting it into the needle valve.

## **4.3 When must the product not be used?**

Contraindications:

The use of the product is prohibited in case of:

- Severe osteoporosis
- Other diseases that greatly reduce the strength of the bony system

Conditional contraindications are defined as follows (only after consultation with physician/therapist):

- Balance disorders
- Dizziness
- Epilepsy.

Other restrictions (expiry of service life see point 3.3.2)

## **5 Cleaning and disinfection**

Clean the product after each use exclusively with solvent-free, non-acidic and non-caustic cleaning agents and disinfectants. The device is waterproof.

## **6 Storage and transport**

Store the product in a place free from sharp and pointed objects to avoid damaging the product.

The material is thermoplastic, which means it softens with increasing heat and solidifies with increasing cold. The product should be stored at room temperature. The product can be stored in a temperature range from -10°C to +45°C.

It must not be stored in direct sunlight (e.g. at the window) or at any other heat source (e.g. at the heater).

Use the supplied packaging when transporting the product.

## **7 Waste disposal**

Products that are no longer used can be returned freight prepaid to

TOGU GmbH  
Atzinger Str. 1  
D-83209 Prien-Bachham

be returned and are fed into the recycling process.

## **8 Legal notice**

The manufacturer accepts no responsibility for personal injury, damage to animals, property or the environment resulting from improper use. Please always follow the instructions and warnings accompanying the product.

Report serious incidents involving the use of the product to the manufacturer and the competent authorities of the Member State.

On the product packaging there is a 6-digit code preceded by the abbreviation LOT. This code indicates the date of manufacture in the format YYMMDD.

In the event of a complaint, this code must be used to identify the batch and the materials used.

Classification: Medical device class 1