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Medical device class 1

Table of contents

1	General information	3
1.1	Definitions	3
1.2	Abbreviations	5
2	Safety	6
2.1	Safety instructions	6
2.2	Product lifetime	6
2.3	Remaining residual risks	6
2.4	Cautions/Warnings	6
3	Device description	7
3.1	Intended use	7
3.2	Device description	8
3.3	Performance description	9
3.3.1	Product performance	9
3.3.2	Product lifetime	9
4	Application	9
4.1	Commissioning	9
4.2	Correct use	10
4.3	When should the product not be used?.....	10
5	Cleaning and disinfection	10
6	Storage and transport	11
7	Waste disposal	11
8	Legal notice	11

1 General information

Do not use the device until you have read and understood the operating instructions.

All information contained in these instructions for use is the property of **TOGU GmbH** (= manufacturer) and may not be reproduced or duplicated without prior permission. The manufacturer reserves the right to change or completely replace this instruction manual at any time and without prior notice.

You should always ensure that you are working with the latest version of the operating instructions. If there is any doubt about the up-to-dateness, contact the manufacturer.

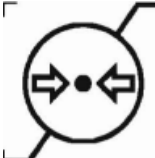
Keep these instructions for use in a safe place and pass them on to other users of the product before use.

TOGU®, Powerball® and ABS® are protected word marks of Gebr. Obermaier oHG.

1.1 Definitions

In these operating instructions, special information is marked as follows:

Symbol	Meaning
	<p>Manufacturer</p> <p>In addition to this symbol, the name and address of the manufacturer, i.e. the person who places the (medical) device on the market, must be indicated</p>
	<p>Date of manufacture</p> <p>Next to this symbol, the date of manufacture of the (medical) product must be indicated in the form YYYY-MM, e.g. 2019-12 for December 2019.</p>
	<p>Usable until</p> <p>For (medical) products that are restricted in their period of use, this symbol shows the date after which the (medical) product may no longer be used in the form YYYY-MM-DD, e.g. 2020-12-01 for 01.12.2020. The date must appear next to the symbol.</p>
	<p>Production lot number/batch</p> <p>Indicates the lot number to which the respective (medical) product belongs. It consists of a 6-digit code and indicates the date of manufacture in the format YYMMDD (year, month, day, each with 2 digits). The lot number is located on the packaging and must be next to the symbol.</p>
	<p>Article number/product identification number</p> <p>Specifies the article or product identification number that must be next to the symbol.</p>

Symbol	Meaning
	<p>Protect from sunlight!</p> <p>If the (medical) product and/or its packaging can be negatively affected in its properties by sunlight, this symbol must be applied to the packaging as a warning.</p>
	<p>Store in a dry place.</p> <p>If the (medical) product and/or its packaging may be negatively affected in its properties by direct moisture, this symbol must be applied to the packaging as a warning.</p>
	<p>Temperature limit.</p> <p>If the (medical) product and/or its packaging can be negatively affected in its properties by certain temperature ranges, this symbol with indication of the lower and upper temperature limits must be affixed to the packaging as a warning.</p> <p>The limit values must be indicated in each case next to the horizontal lower or upper line.</p>
	<p>Air humidity limit.</p> <p>If the (medical) product and/or its packaging can be negatively influenced in its properties by certain air humidity ranges, this symbol with indication of the lower and upper air humidity limits must be affixed to the packaging as a warning.</p> <p>The limit values must be indicated in each case next to the horizontal lower or upper line.</p>
	<p>Air pressure limit.</p> <p>If the (medical) product and/or its packaging can be negatively influenced in its properties by certain air pressure ranges, this symbol with indication of the lower and upper air pressure limits must be affixed to the packaging as a warning.</p> <p>The limit values must be indicated in each case next to the horizontal lower or upper line.</p>
	<p>Observe the instructions for use!</p> <p>If prior reading of the operating instructions is required for correct use of the (medical) product, this symbol must be clearly visible.</p>
	<p>"Reference to eIFU" shall be replaced with the website address where the user can find the electronic user documentation.</p> <div style="display: flex; justify-content: space-around; align-items: center;"> <div style="text-align: center;">  Hinweis auf eIFU </div> <div style="text-align: center;">  Hinweis auf eIFU </div> <div style="text-align: center;"> Hinweis auf eIFU  </div> </div>
	<p>Attention!</p> <p>If it is absolutely necessary to read the operating instructions before using the (medical) product without risk, this symbol must be clearly visible.</p>
<p>Max. xxx kg</p>	<p>Performance information relevant for the safe use of the product</p>
	<p>MD = Medical Device</p> <p>Marking that the product is a medical device.</p>

Symbol	Meaning
	<p>CE conformity marking (minimum height 5mm). Indicates compliance with the Essential Requirements of the applicable European Directive/Regulation. For Class I medical devices, only the two letters are affixed. For medical devices > Class I, the two letters are followed by the four-digit number of the Notified Body that carried out the conformity assessment procedure. For medical devices > Class I, the two letters are followed by the four-digit number of the Notified Body that carried out the conformity assessment procedure.</p>

1.2 Abkürzungen

Shortcut	Meaning
ABS®	Anti Burst System
JJMMTT	Year (2 digits), Month (2 digits), Day (2 digits)

2 Safety

2.1 Safety instructions

2.2 Product Lifetime

If the service life of 2 years from first use, or at the latest 10 years after manufacture (see lot number) is exceeded, safe use of the product is no longer guaranteed. Replace the product immediately.

2.3 Remaining residual risks

Please keep the exercise field free of sharp objects or edges. Before each use of your product, check that the surface is undamaged. Product safety is no longer guaranteed if the surface is damaged or if the safety limits described in this manual are exceeded.

2.4 Cautions/Warnings

The product must not be misused, modified or changed in any other way. Safe use is no longer guaranteed in this case.

Only use the product on a stable, horizontal, dry and clean surface. In this way, prevent the product from slipping away.

Keep the product away from:

- sharp and pointed objects
- fire or other sources of heat
- Chemicals

Do not wear clothing or shoes with sharp or pointed parts while using the product.

Keep the area around the product free of objects (furniture, fixtures) within a radius of the user's body length. Keep your distance from other persons. This will prevent injuries in the event of a fall.

The product is unstable. You may suddenly lose your balance. Move onto the product carefully and slowly. Use safe, firmly anchored aids to avoid a fall and injuries.

Sweating or moisture from outside (e.g. rain) can make the product slippery. Also check the surface of the product during use. In this way, avoid slipping and injury in the event of a fall.

The product should be used at room temperature. The product can be used in a temperature range from **-10°C to +45°C**. It must not be used in direct sunlight (e.g. by the window) or near any other heat source (e.g. by the heater).

The material is thermoplastic: this means that it becomes softer with increasing heat and firmer with increasing cold.

Check the product for damage, wear, and functionality before use. If there are any deviations, the product must no longer be used. Remove and replace the product if it shows signs of damage. Do not attempt to repair the product.

If the instructions for use are not followed, an MD Powerball Premium ABS® - like any other ball - can burst if it hits a pointed or sharp object.

Seek professional advice (doctor, therapist) before performing exercises with the product. Children, elderly or impaired persons may use the product only under the supervision of an adult, physician or therapist familiar with the product.

In these cases the product should not be used: see section 4.3

3 Device description

MD Powerball Premium ABS

The MD Powerball Premium ABS® balls are made of anti-burst material and offer you increased safety against sudden bursting of the ball when in use, provided that a certain body weight is not exceeded (see 3.3.1 Performance of the product). Due to the special material composition (Pat. No.: 44 34889.4), the MD Powerball Premium ABS® can no longer burst in case of sudden punctual damage, but only slowly loses its air.

The anti-burst material therefore protects you from sudden bursting of the ball in case of punctual damage or stitches. Instead, the ball loses air so slowly that you have enough reaction time. However, in the case of larger lacerations, e.g. abrasions on sharp edges and the associated tearing of the ball skin by several centimeters, even the anti-burst material can no longer prevent the ball from suddenly losing air = bursting in the event of such an injury.

For this reason, the ball should be inspected regularly for scratches or scuffs that may occur or gradually develop over time, even when used as intended.

Characteristics of the Anti-Burst Material:

The anti-burst material may cause the shape of the ball to change slightly during use. However, this has no influence on the dynamics or the movement sequence of the exercises, because the core benefit of active dynamic sitting, i.e. also all movements performed while sitting on the ball, lies in the mediation of balance-sensitizing stimuli. The decisive factor is therefore the lability of the ball, not its circular nature. Approximately ten minutes after leaving the ABS® ball, the ball regains its round shape due to the restoring force of the material. Due to the packaging, an ABS safety ball has some wrinkles when inflated. Due to the high material strength, these wrinkles may also be permanently visible on an inflated MD Powerball Premium ABS®. This in no way affects the functionality or safety.

All balls manufactured by us can be recycled several times, therefore a defective ball represents a high-quality raw material. Please recycle a defective ball in order to exhaust ecological resources.

Our products comply with the German Foodstuffs and Commodities Act (BGVV) and do not contain any heavy metals.

In addition, we guarantee that our products are free of harmful or odorous vapors.

3.1 Intended use

The MD Powerball® Premium ABS is a medical device and is used as a non-active therapeutic device for::

- Diseases of the musculoskeletal system,
- as prophylaxis in the professional field and
- as prophylaxis in the private/amateur sector.

The product is used by physiotherapists (or similar qualification) in rehabilitation or physiotherapy facilities (or similar), as well as by patients (age from 6 years) in the home environment.

In therapy, the MD Powerball Premium ABS® is used, among other things, in orthopedics and neurological rehabilitation to train mobility, strength and coordination and also serves to promote sensory perception, coordination and the musculoskeletal system.

Contraindications :

The ball has been developed for various therapeutic and sporting applications and must not be used other than for the specified purpose.

The use of the product is prohibited in case of:

- Severe osteoporosis
- Other diseases that greatly reduce the strength of the bony system

Conditional contraindications are defined as follows (only after consultation with physician/therapist):

- Balance disorders
- Dizziness
- Epilepsy

Do not train with weights or dumbbells of any kind on the MD Powerball Premium ABS®.

3.2 Device description

Description	Size	Weight net	Recommendation for body size
MD Powerball® Premium ABS® 55 cm	max. 55 cm	1400 g	156 cm – 165 cm
MD Powerball® Premium ABS® 65 cm	max. 65 cm	1550 g	166 cm – 178 cm
MD Powerball® Premium ABS® 75 cm	max. 75 cm	1600 g	ab 178 cm

The variant of the product is marked on the packaging.

The product contains two valve caps made of plastic.

The package includes packaging, quick start guide, instructions with examples of use.

3.3 Performance description

In the table below, the load capacity is given in relation to the maximum load capacity, i.e. how much weight an MD Powerball Premium ABS® can safely support in undamaged condition (max. load).

In addition, the load is listed up to which the product is guaranteed to be space-safe even in the event of damage.

3.3.1 Product performance

Description	max.load	Space-saving up to load
MD Powerball® Premium ABS ® 55 cm	ca. 1000 kg	170 kg
MD Powerball® Premium ABS ® 65 cm	ca. 1000 kg	170 kg
MD Powerball® Premium ABS ® 75 cm	ca. 1000 kg	120 kg

3.3.2 Product lifetime

If the service life of the product exceeds 2 years from the date of first use, or a maximum of 10 years from the date of manufacture (see lot number), safe use of the product is no longer guaranteed. Replace the product immediately.

4 Application

4.1 Commissioning

When opening the packaging, take care not to damage the product.

Check the product for damage, wear and functionality before use. If there are any deviations, the product must no longer be used. Remove and replace the product if it shows any damage. Do not attempt to repair the product.

Follow the steps below to use the product and activate the anti-burst material.

The ball must have been stored for two to three hours at a minimum of 20°C before inflation.

- 1) Inflate the tempered ball in stages at room temperature to about 80% of the maximum size. Note that the ball can be very hard at this stage. Allow to rest for at least two hours.
- 2) Slowly inflate the ball at room temperature or outside temperatures of max. 25°C to the optimal size for you, but not over the maximum size. Attention: due to the leather-like nature of the material, the ball feels very hard even when inflated slowly. The ball will reach its optimal elasticity (softness) after two days at the latest. Since the leather-like ABS® material naturally needs more time to get used to stretching than a conventional, non-reinforced ball, it may tear if not handled properly during inflation.
- 3) It is essential to let balls with ABS® rest for 24 hours after final inflation before first use, so that the anti-burst material can adapt to the external conditions (optimum room temperature). Please check the diameter of your ball again after these 24

hours, as the ball may become larger due to air expansion inside the ball as a result of temperature exposure or adjustment. If the maximum diameter has been exceeded due to this, simply deflate a little to achieve the correct diameter.

4.2 Correct use

Observe the detailed safety instructions under point 2.

The product should be used at room temperature. The product can be used in a temperature range from **-10°C to +45°C**. Keep the area around the product free of objects (furniture, fixtures, etc.) within a radius of the user's body length. Keep a distance from other persons. This will prevent injury in the event of a fall.

Before each use, check the product for damage, wear and tear and proper functioning. If there are any deviations, the product must no longer be used. Remove and replace the product if it shows signs of damage.

If the service life of the product exceeds 2 years from the date of first use, or a maximum of 10 years from the date of manufacture (see lot number), safe use of the product is no longer guaranteed. Replace the product immediately.

For the application, please note:

- Adopt a correct posture
- Perform slow and controlled movements
- If pain occurs, seek advice from a doctor or therapist one
- Children are allowed to train only under the supervision of adults

4.3 When must the product not be used?

Contraindications:

The use of the product is prohibited in case of:

- Severe osteoporosis
- Other diseases that greatly reduce the strength of the bony system

Conditional contraindications are defined as follows (only after consultation with physician/therapist):

- Balance disorders
- Dizziness
- Epilepsy
-

Other restrictions (expiry of service life see point 3.3.2)

5 Cleaning and disinfection

Clean the product after each use exclusively with solvent-free, non-acidic and non-caustic cleaning agents and disinfectants. The device is waterproof.

6 Storage and transport

Store the product in a place free from sharp and pointed objects to avoid damaging the product.

Da he material is thermoplastic, which means it softens with increasing heat and solidifies with increasing cold. The product should be stored at room temperature. The product can be stored in a temperature range from -10°C to +45°C.

It must not be stored in direct sunlight (e.g. at the window) or at any other heat source (e.g. at the heater).

Use the supplied packaging when transporting the product.

7 Waste disposal

Products that are no longer used can be returned freight prepaid to

TOGU GmbH
Atzinger Str. 1
D-83209 Prien-Bachham

be returned and are fed into the recycling process.

8 Legal notice

The manufacturer accepts no responsibility for personal injury, damage to animals, property or the environment resulting from improper use. Please always follow the instructions and warnings accompanying the product.

Report serious incidents involving the use of the product to the manufacturer and the competent authorities of the Member State.

On the product packaging there is a 6-digit code preceded by the abbreviation LOT. This code indicates the date of manufacture in the format YYMMDD.

In the event of a complaint, this code must be used to identify the batch and the materials used.

Classification: Medical device class 1