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Instruction for use Dynair® Walker Comfort

Congratulations! You are now the owner of a pair of Dynair® Walker Comfort, a product “made in Germany”. This sports tool was produced in compliance with valid EU norms, it contains no heavy metals and is made of 100% recyclable plastic.

The innovative and uncomplicated training tool is developed for training and stabilisation your ankle joint. The reactive training allows you during the exercises a free moving upper body. So you can increase your training effect with extra exercises for arms or torso. The most common injuries in ball sports concern the ankle joint. Here, the device can be optimally used in rehabilitation. Especially suitable for preventive measure to protect your ankle joint and minimize injury risk. Preventive training is even for leisure and professional sporting useful. The Dynair Walker Comfort also can be integrated in your home training e. g. to approach chronic instability of your ankle joint. A detailed training- and usage instruction is enclosed.

Read the following safety instructions and use the product only as described to avoid injuries or damages. Keep this manual for later reference. Forward the instructions to every user.

How to inflate:

The air filling of the Dynair Walker Comfort can be regulated by using the attached TOGU ball pump. Therefore the Dynair Walker are equipped with a needle valve. Please perform changes to the air pressure inside the Dynair Walker (inflation or bleeding) only, when the Dynair Walker has room temperature and add at the most as much air, as is necessary to make the surface level. Always moisten the needle before carefully inserting into the needle valve.

Do not over-inflate: max. height is 7cm. Risk of break-up or bursting.

Adjustment:

Please follow these steps as described below:

1. Place your foot in a central position on the black base plate. Small shoe sizes place in front, larger ones on the back line.
2. Tighten the heel strap first and then tighten the wristband.
4. Also tighten the last band at the forefoot. Check if the foot is fixed correctly.
5. Control the correct central position of your foot on the Dynair Walker Comfort.

Warning:

- Avoid storage in direct sunlight or near heat sources, because otherwise the Dynair Walker Comfort might deform
- Avoid contact with sharp and pointed objects (e.g. broken glass, pointed rocks in shoe soles) because they can damage the Dynair Walker Comfort. Do not throw that item.
- Use sport shoes, regular shoes or be barefoot while training on the Dynair Walker Comfort.
- Prior to any use, the product should be checked for damage because otherwise a safe handling is not guaranteed
- The Dynair Walker Comfort have to rest on a horizontal, stable and dry ground, to make sure, that the product cannot slide away during the exercises.
- Please do not use the item other way round, e.g. stand on or put weight on the red, air-filled part, because the plate could be damaged

Equipment details:

The Dynair Walker Comfort is made of high grade Ruton with a needle valve and comfortable soft neoprene buckles. Size: one size -fits all: EUR 37 to 46, Max. Load: approx. 150 kg

Care:

Use mild standard detergents and disinfectants. Dissolvers as well as acidic or corrosive cleansers can harm the material. The Dynair Walker Comfort are waterproof.