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## Massage-Nex / Nex-Neckrest Usage and Care Instructions

**Congratulations on your purchase of the Massage-Nex / Nex-Neckrest.**

**Having purchased the Massage-Nex / Nex-Neckrest you have chosen a TOGU® branded product which is "Made in Germany"!**

**It is made in conformity with the applicable EU regulations for sports equipment and is free of banned phthalates and heavy metals.**

**Please read the instructions carefully.**

### **Important!**

If you are experiencing problems in the cervical spine area, please consult your physician or therapist before use. The Massage-Nex / Nex-Neckrest is no substitute for therapy.

### **Application:**

Lie down with your head on the Massage-Nex / Nex-Neckrest so that the lower bones of the skull are comfortably supported. You will feel how your neck musculature relaxes. Remember, the head must be moved during use.

### **Duration of application:**

Up to about 10 minutes, maximum duration as long as you feel comfortable.  
Once or twice a day.

### **Equipment details:**

The Product is made of high-quality Ruton and is fitted with an inflation valve.

Measure: 20 x 10,5 cm

Load capacity:

Massage-Nex: approx. 150 kg

Nex-Neckrest: approx. 200 kg

### **Thermal stability:**

The Massage-Nex / Nex-Neckrest is UV-resistant and also heat-resistant to a limited degree. The can easily withstand temperatures of up to 45°C. However, the Product should be kept out of direct sunlight (e.g. do not store in the window). The material softens as it gets warmer.

The Massage-Nex / Nex-Neckrest is also resistant to cold, but should nevertheless not be stored at temperatures below -10°C. The material hardens as it gets colder.

### **Care:**

Any mild standard cleaning agents and disinfectants can be used to clean the Massage-Nex. / Nex-Neckrest Solvent-based, acidic or caustic cleaners will damage the material.

The Massage-Nex / Nex-Neckrest is waterproof.